Clinical Results

Rosacea



Port Wine Stain



Wrinkles



Brown Pigmentation



eline Post 1 Treatment

Courtesy of T. Kono, M.D.

Trust the Vbeam Perfecta laser to treat:

- ☑ Redness of the face and neck
- ☑ Rosacea
- ☑ Sun spots
- ☑ Age spots
- ☑ Facial and leg veins
- ☑ Freckles
- ☑ Scars and stretch marks
- ☑ Warts
- ☑ Wrinkles
- Psoriasis
- ☑ Poikiloderma

Schedule a personal consultation with your practitioner today!

candelamedical.com

© 2020 Candela Corporation. This material contains registered and unregistered trademarks, trade-names, service marks and brand names of Candela Corporation and its affiliates. All other trademarks are the property of their respective owners. All rights reserved. PB75941EN-NA, Rev. A



Redness, Brown Spots, Wrinkles, Vascular Lesions



The Gold Standard in Vascular Treatments



Vbeam Perfecta®

Proven results delivered with best in class technology

Trust the Vbeam Perfecta system to successfully remove discolorations, including rosacea, sun spots, age spots, freckles, as well as other vascular and pigmentation abnormalities such as poikiloderma, port wine stains, facial veins and leg veins.

In addition, the Vbeam Perfecta device is proven to treat scars, stretch marks, warts and psoriasis. The Vbeam Perfecta system uses Pulsed Dye Laser (PDL) technology to deliver an intense but gentle burst of light into targeted areas of the skin. There it's absorbed by the blood vessels or melanin pigmented areas, removing the targeted imperfections. Most patients notice results right away, and it's clinically proven for multiple skin types.

What is the Vbeam treatment like?

Most patients are reclining or lying down, depending on the area being treated. The laser will be calibrated and parameters set based on the specific condition being treated. Patients often report a warming or tingling sensation during the treatment, although anesthetics are usually not required.

Are the treatments painful?

Most patients report a sensation similar to mild sunburn after the treatment. During the treatment, skin may receive additional protection through the Dynamic Cooling Device cooling system (DCD™), which delivers a cooling burst of cryogen before the laser pulse is delivered. The DCD cooling increases comfort during treatment and minimizes side effects such as redness.

Are there any side effects?

PDL treatments have been used since the 1980s for the treatment of port wine stains in infants and young children. The DCD cooling goes a long way toward ensuring both the patient's comfort and the protection of the treated skin. Some patients experience redness immediately after treatment, but it usually goes away in a few hours. Occasionally a laser bruise called purpura may occur; this usually disappears in three to five days.

Are PDL treatments permanent?

Permanency depends on several factors, including the condition being treated and the patient's lifestyle. Sun spots, for example, may return if the patient continues to spend time in the sun without protection. Talk with your practitioner about your goals and expectation for treatment.

Is Vbeam PDL treatment covered by insurance?

Most insurance carriers do not offer reimbursement for cosmetic and aesthetic procedures. Some carriers will, however, cover the treatment of disfiguring birthmarks such as hemangiomas or port wine stains, and some will cover psoriasis and rosacea. Please consult your insurance carrier for details.

How much do treatments cost?

Cost depends on the condition and size of the area being treated. Discuss your personal needs with your provider to learn more.

"I was nervous during my first treatment for sun spots because I wasn't sure what to expect. It was over before I knew it and I didn't feel a thing—and the results have been great!"

S.L., California

